

ROLLATOR USER GUIDE INTRODUCTION

ALJ-009A-GRAY FRAME	ALJ-009B-RED FRAME	ALJ-008H-BLUE FRAME
		

All these 4-Wheeled Rollator Walkers are intended to be built lightweight. They are designed to be portable as well as to aid user to walk, and as an option, to carry some personal items. They come with either wire basket or fabric pouch.

These rollators are suitable for indoor and outdoor use. They provide a cost-effective solution to some mobility problems whilst offering stability and security to the user.

USER GUIDE INSTRUCTIONS

1. To open your rollator, push the folding hinge (side braces) down until it locks securely in a straight position.
2. Place both loose hand grips, with the pre-assembled deluxe loop brakes, into the handlebar openings and adjust the desired height. To do this, remove the angular knob, washer and screw from the handle bar. Insert, the screw, with the **washer** on it, through the desired hole aligned in the rollator frame-rod and screw the angular knob back on securely.
3. Insert the removable backrest into rollator frame securely by making sure the retainer pins are locked into position through the frame hole.
4. Test and adjust the tension on the braking system mechanism to ensure proper and secure braking system before use.
5. Secure the wire basket or fabric pouch in the proper location. Flip up the seat and place the basket on the rollator crossbars or position the wire basket hooks over the front bar (not shown).
6. Always ensure the side braces are fully down in full extension before using.

7. People who may have hand disabilities should find the brake handles comfortable and non-binding. Minimum pressure will need to be applied to activate the locking loop brake system.

SPECIFICATIONS

ALJ-009A-GRAY FRAME	ALJ-009B-RED FRAME	ALJ-008H-BLUE FRAME
Overall Product Dimensions:		
53 (W) x 68 (L) x 94 (H) cm	58 (W) x 63 (L) x 95 (H) cm	53 (W) x 68 (L) x 94 (H) cm
Weight Capacity:		
120 Kg. (Stationary); 80 Kg. (Kinetic)	120 Kg. (Stationary); 80 Kg. (Kinetic)	120 Kg. (Stationary); 80 Kg. (Kinetic)
Handle Height:		
87-94 cm	83-95 cm	87-94 cm
Seat Width:		
36 cm	40 cm	36 cm
Environmental conditions:		
Between 45°C & -5°C	Between 45°C & -5°C	Between 45°C & -5°C
Function:		
Semi-automatic & Foldable	Semi-automatic & Foldable	Semi-automatic & Foldable

IMPORTANT SAFETY INFORMATION

Please read all these instructions fully before assembly or use:

1. For safety reasons, the parking brake should always be engaged before sitting down onto the seat.
2. Do not push rollator when the brake is engaged.
3. Do not use rollator seat as step stool.
4. Wire basket or fabric pouch should be removed before folding any rollator.
5. Do not lift up the back wheels of the rollator and push it forward using only the two front wheels.
6. Do not use any volatile solution such as thinner or benzene cleaning products. Use tap water to clean your rollator after use. Use a cleaning rag, towel or terry cloth to remove water stains.
7. To preserve product integrity, store your rollator in a dry and well ventilated place; avoid extreme heat, cold and moisture.
8. If the function of the braking system cannot be activated for any reason, it can still be activated manually on both sides of the wheels.